# AQUISEL

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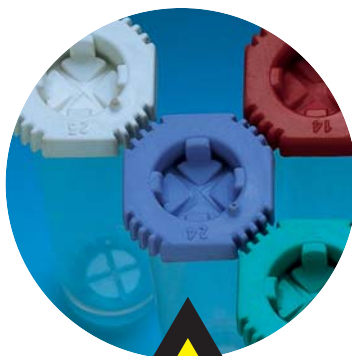
*Eliminate unnecessary risks with the AQUISEL Valve-Stopper system.*

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*Container with AQUISEL Valve-Stopper.*

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*Fixation and concentration of faecal parasites "Aquisfek"*

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*Pipettes for E.S.R. with zero levelling and automatic closure system for conventional tubes or AQUISEL Valve-Stopper.*

EN-ISO-9001 SISTEMAS DE GESTIÓN DE LA CALIDAD / EN-ISO-13485 PRODUCTOS SANITARIOS  
EN-ISO-9001 QUALITY MANAGEMENT SYSTEMS / EN-ISO-13485 MEDICAL DEVICES

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### **AQUISEL VALVE-STOPPER**

**System available for sample manipulation without the need of uncorking the tube, maintaining it hermetically sealed and avoiding the risk of contamination.**

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The AQUISEL system consist of a tube, valve-stopper and an appropriate reagent for specific use.

The AQUISEL tube is made of a high resistance transparent polypropylene. The valve-stopper is a very elastic material and **has a special cross-cut design which allows an automatic recover to its initial closed position, after insertion or extraction of the sample.**

The tube's reagent contents is offered through different possibilities identified by colours in accordance with International Standards.

It can be used by most of the current analytical procedures, even if they are manual, semi-automatic or automatic procedures.

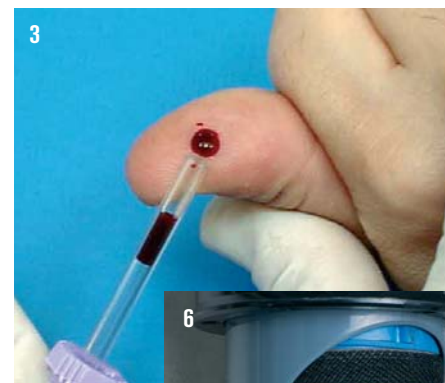
It allows inter-hospital road and postal transport thanks to its hermetic seal and high resistance to uncorking due to vibrations, hits, cold, heat, etc.

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### **USE AND APPLICATIONS**

1. Inserting (blood) samples with syringe (without needle).
2. Extraction and dosage with semi-automatic pipette.
3. Capillary blood extraction.
4. Capillary filling for microhematocrite.
5. Automatic sampling for haematology in primary tubes.
6. Sample road transport.
7. Centrifuge heads adaptable.

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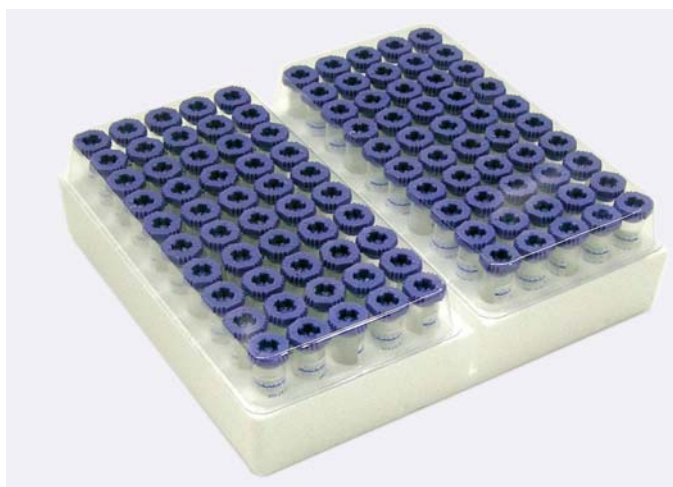
There are different size and volume tubes with additives: anticoagulant, antiglycolitics, clotting accelerators or empty tubes.

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Suitable for paediatrics or small sample volumes.

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The tubes are presented in polystyrene racks of 100 units. Every 50 units are covered with a transparent lid for protection.

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Funnel adapter which can be used together with the primary AQUISEL tubes and allows sample blood transfer to secondary tubes, useful for capillary blood extractions in paediatrics, for low blood volumes, can be used in all auto-analysers for primary tube sampling, etc. All these procedures could be completed without uncorking the primary AQUISEL tubes. Once the CAP-GALET is removed, the tube remains sealed again for conservation or disposal.

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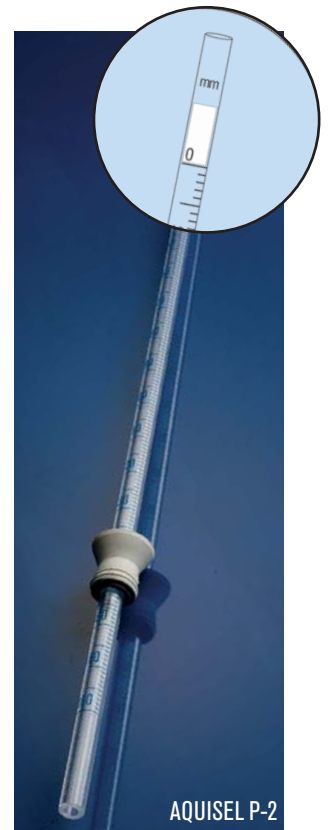
## AQUISEL PIPETTE

PIPETTES FOR ERYTHROCYTE SEDIMENTATION READ E.S.R. WESTERGREN METHOD.

Pipettes made in polystyrene with similar properties as glass ones.

The ZERO-CLOSURE system allows the pipette filling, preventing that the blood exceeds the zero point and also avoiding total or partial loss. Automatic sealing makes it a disposable product.

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### AQUISEL Pipette A-2 without rubber.

For any kind of tube use, even with AQUISEL or with a conventional stopper.

Suitable for mechanical or manual aspiration.

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### AQUISEL Pipette P-2 with rubber.

For 75x13 tubes with AQUISEL or with a conventional stopper.

Pipette filling by pressure till the bottom of the tube.

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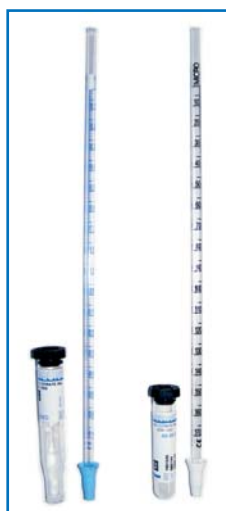
### AQUISEL Pipette P-3 and P-4 MICRO systems

Set of AQUISEL pipettes and tubes. Tubes come with the valve-stopper which allows the pipette insertion without the need of uncorking.

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### AQUISEL Pipettes A-2

Expanded polystyrene packaging of  
500 units.

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### AQUISEL Pipettes P-2

200 units cardboard package.  
Supplied in 800 units boxes.

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### Set P-3. AQUISEL Pipette and tube

200 units cardboard package and  
100 tubes rack made of expanded  
polystyrene, supplied in 600 sets  
boxes.

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### Set P-4 Micro. AQUISEL Pipette and tube

100 pipettes cardboard package and  
100 tubes rack made of expanded  
polystyrene, supplied in 400 sets box.

MODELO MODEL	CÓDIGO CODE	Nº UNIDADES CAJA UNITS / BOX	PESO CAJA Kg BOX WEIGHT Kg	MEDIDAS CAJA cm - BOX DIMENSIONS cm		
				Ancho Width	Fondo Depth	Alto Height
A-2	1750224	500	1,5	27	14	11
P-2	1780225	800	4	55	27	26
Conjunto P-3 Set P-3	1760200	600 pipetas + 600 tubos 600 pipettes + 600 tubes	6	47	43	26
Conjunto P-4 Micro Set P-4 Micro	1740100	400 pipetas + 400 tubos 400 pipettes + 400 tubes	3,6	20	40	37

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### Metallic Westergren stand

10 place autoclavable stand.

For any kind of AQUISEL pipettes with Ø 13 mm  
tubes and A-2 pipettes with Ø 16 mm tubes.  
Easy and functional system that allows the total  
set disposal.

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### AQUISEL URINE CONTAINER

CONTAINER FOR URINE SAMPLES COLLECTION WITH AQUISEL VALVE-STOPPER.

The AQUISEL VALVE-STOPPER allows the urine manipulation without uncorking, eliminating the contamination risks, samples loss, spillage, odours, etc. All the AQUISEL urine container presentations will be supplied with the GALET accessory, which makes easy the sample dosage to a tube for sedimentation analysis, or the urine test strip insertion.

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Insertion of the GALET in the AQUISEL VALVE-STOPPER.

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Urine sample extraction through the GALET.

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Dosage through automatic or conventional pipette.

CÓDIGO CODE	BOLSA BAG	MONTADO ASSAMBLED	UDS. CAJA UNITS BOX	PESO Kg WEIGHT Kg	MEDIDAS cm - DIMENSIONS cm		
					Ancho Width	Fondo Depth	Alto Height
1660223	NO NO	NO NO	600	12	60	40	43
1640234	NO NO	SI YES	400	8	60	45	43
1630233	SI YES	SI YES	300	7	60	45	43

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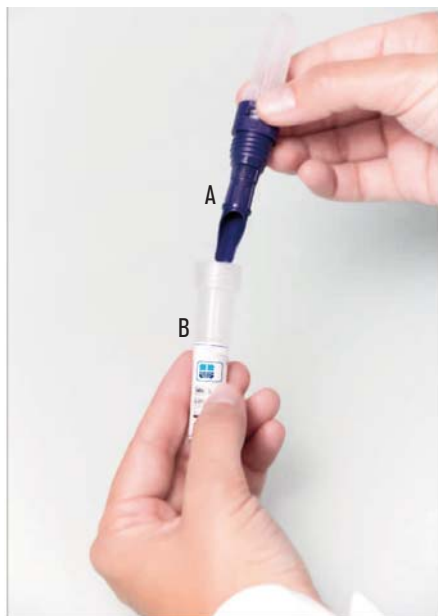
## FIXATION AND CONCENTRATION OF FAECAL PARASITES “AQUISFEK”

New compact design of Aquisfek device, which allows an easy and convenient sample collection, a safe transportation with an easier handling system and a more effective filtration.

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#### SAMPLE COLLECTION.

#### EASY TO HANDLE BY THE PATIENT.

Separate the two parts (A and B) of the device and with the aid of the spoon built-in at the end of the filter, collect the sample.

Introduce the spoon with the sample back into the liquid and screw to close.

#### PROCESSING THE SAMPLES IN THE LABORATORY.

#### FAST AND EASY.

Once the sample has been homogenized, the whole device is centrifuged.

After centrifugation, discard the part A (spoon) and with the aid of a pipette, remove the supernatant in the tube (Part B).

The concentrated sample residue is transferred to a slide for a further microscopic observation.

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#### TWO DIFERENT FORMATS

**KKIT 1.** With formalin / acetate. Parasite common fixer.

**KIT 2.** Without formalin to minimize the risk of environmental contamination.

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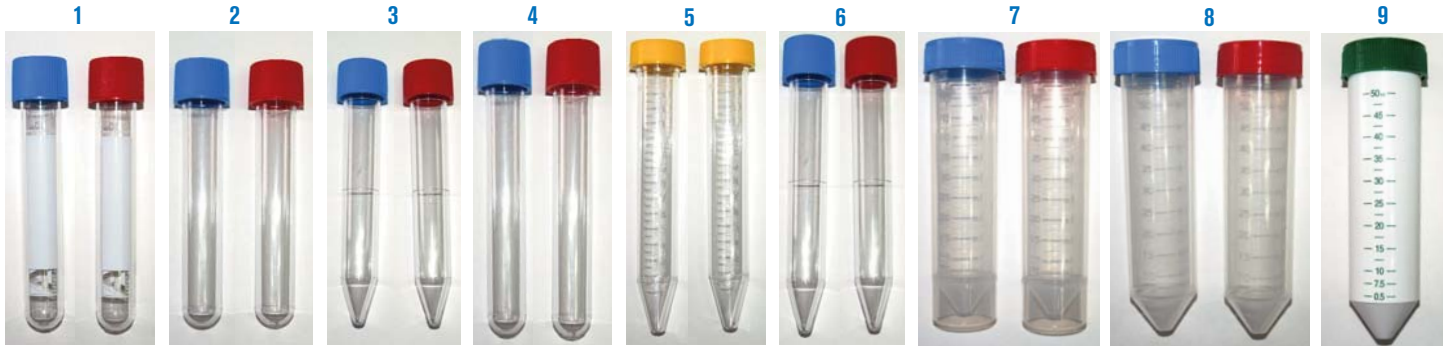
**Note:** For centrifugation, Cencom 2 is recommended. Part number 7002240. See page 114

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